




Urine Monovette® - User Guide

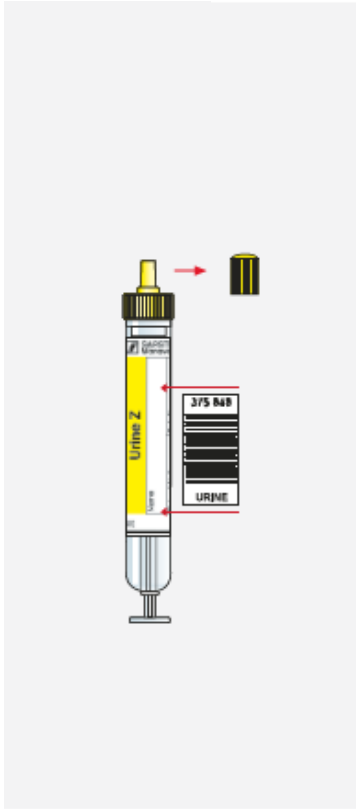


When using a barcode label, attached it along the printed line and/or place it in a **exact vertical** position. **Remove stopper and keep for later use!**

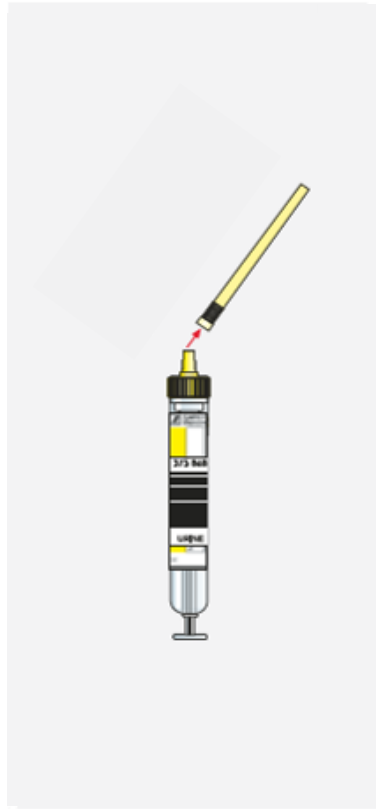


Disinfect the septum in accordance with internal guidelines. Insert the Luer tip into the septum. Attach aspiration tip.

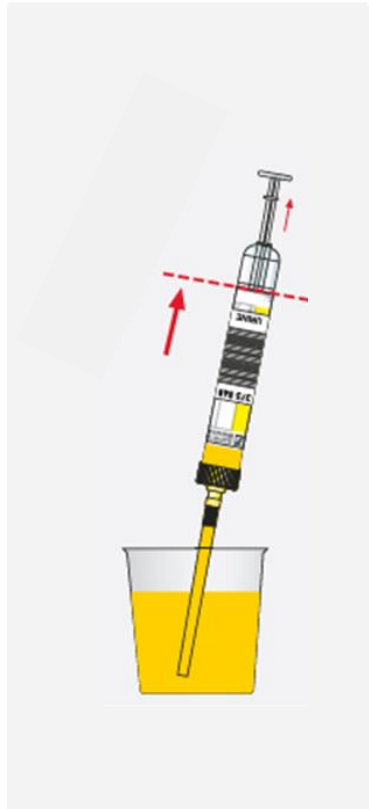

Insert the aspiration tip into the container and **fill the Urine Monovette® up to base line.**

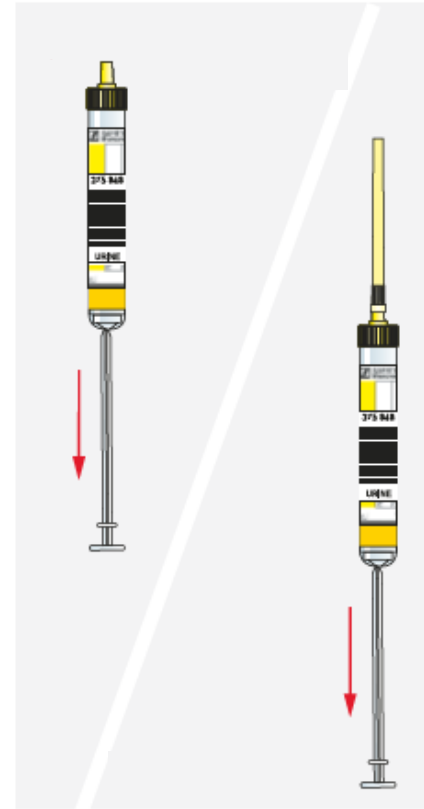

Hold the Urine Monovette® in a vertical position and withdraw the piston rod in downward direction until it locks into its base to empty the aspiration tip.

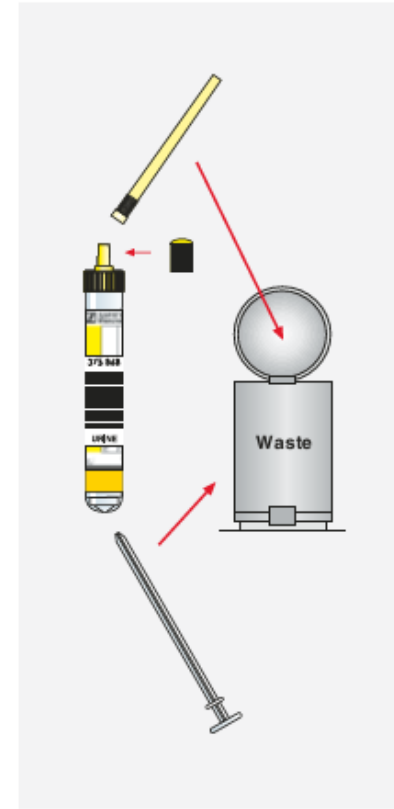

Remove the aspiration tip, **break off the piston rod and discard both.** **Replace the stopper.**

Please follow the bold printed notes.